

Advanced Vocabulary Creation and Customization in Proloquo2Go for iOS

David Niemeijer & Jennifer Marden
AssistiveWare®

Agenda

- Introduction
- Unique customization techniques in Proloquo2Go 3
- New customization techniques in Proloquo2Go 4
- Q&A

Financial Disclosure

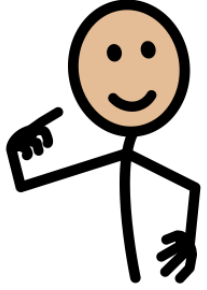

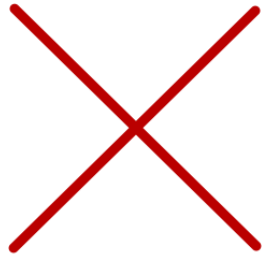
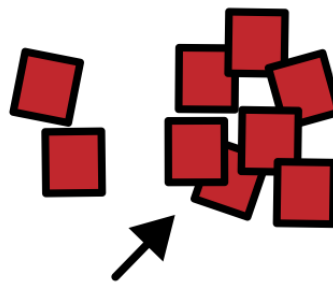
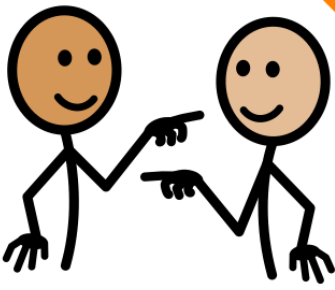
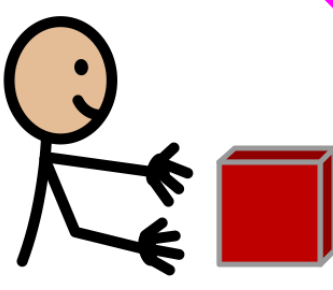
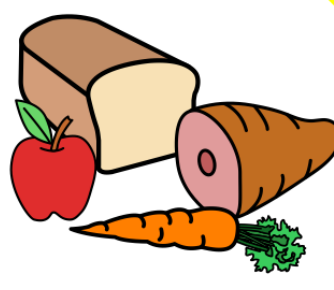
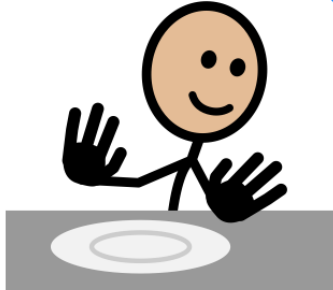
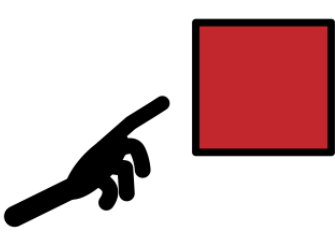
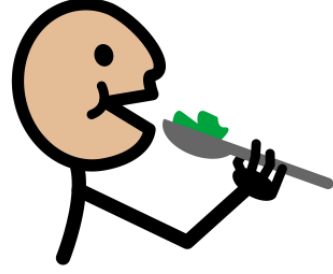
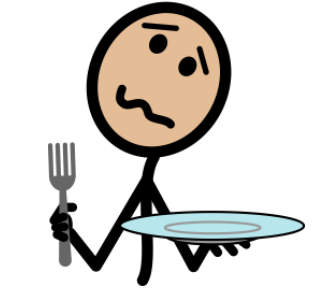
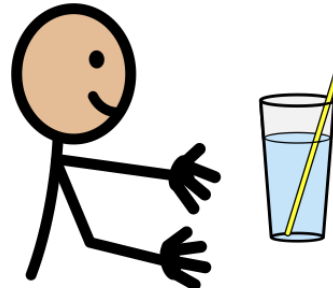
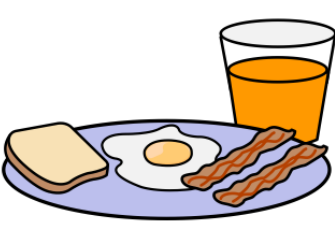
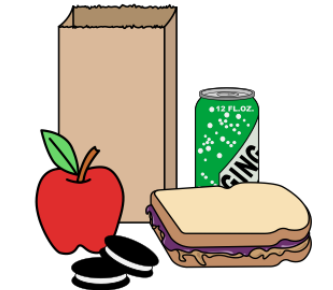

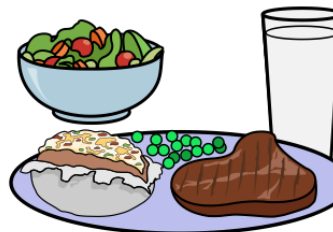
- Both presenters are employed by AssistiveWare, developers of Proloquo2Go

Proloquo2Go Features

Feature	Version 3	Version 4	Benefit
VocaPriority™	✓	✓	Quick customization
Resizable Grid	✓	✓	Gradual growth
Auto-inflections	✓	✓	Language development
Multi-level		✓	No re-customization
Live templates		✓	Motor planning
Multilingual		✓	Support bilingual speakers
Popups		✓	Contextual choices
Snapback		✓	Automatic navigation

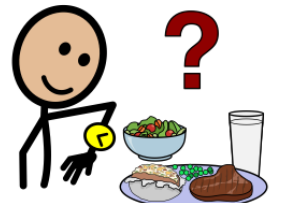
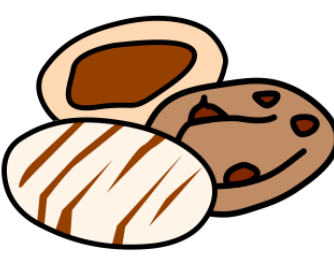
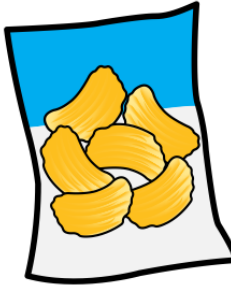

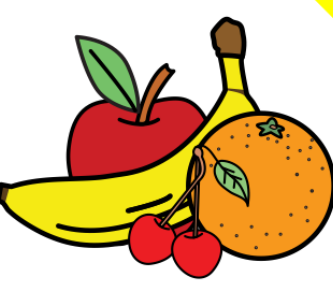
Vocabulary Levels

Things Food

 I	 what	 not	 more
 you	 want	 food	 all done
 it	 eat	 I'm hungry	 I'm thirsty
 Breakfast	 Lunch	 Snacks	 Dinner

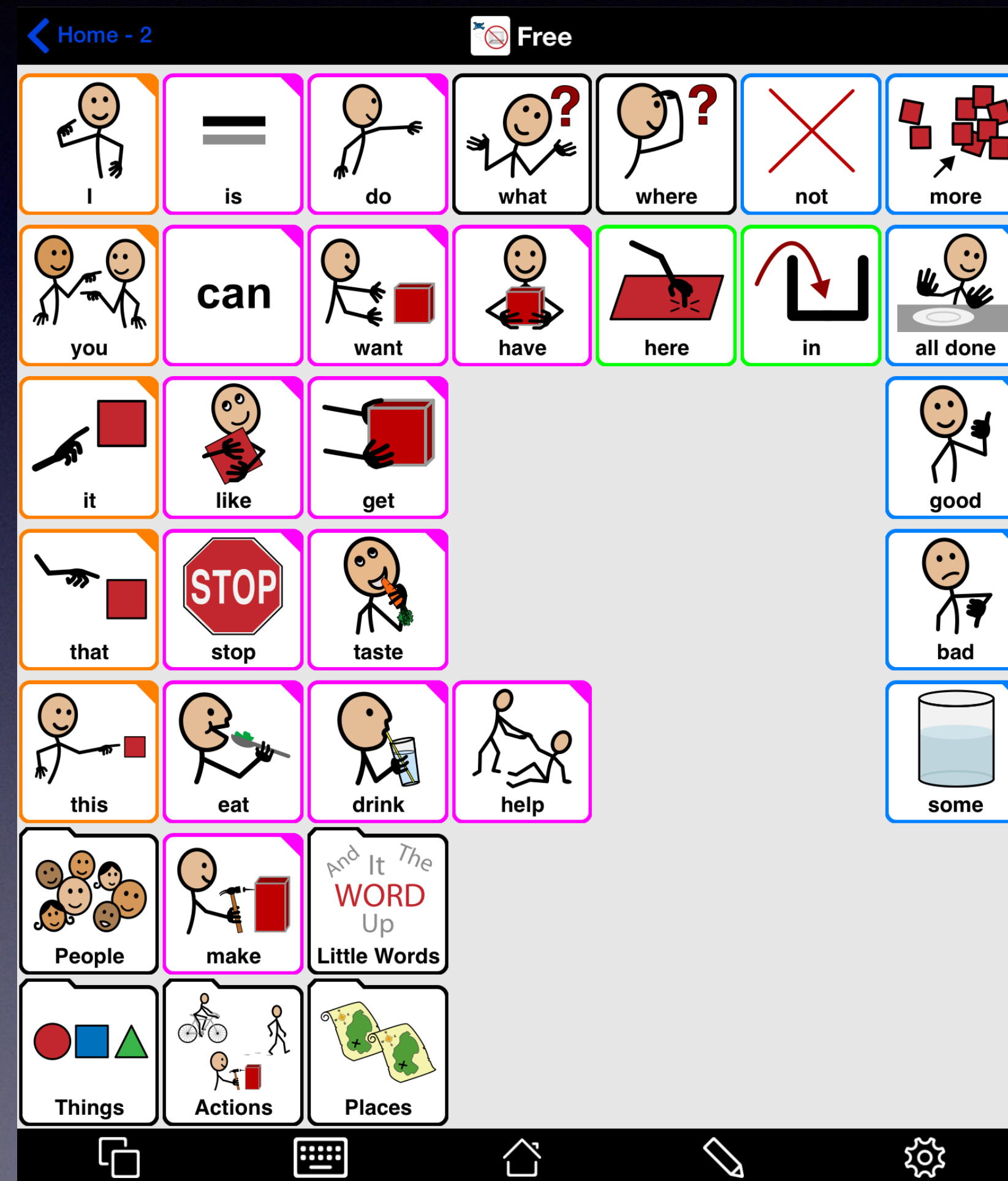
Navigation icons: copy, keyboard, home, edit, settings

Things Food

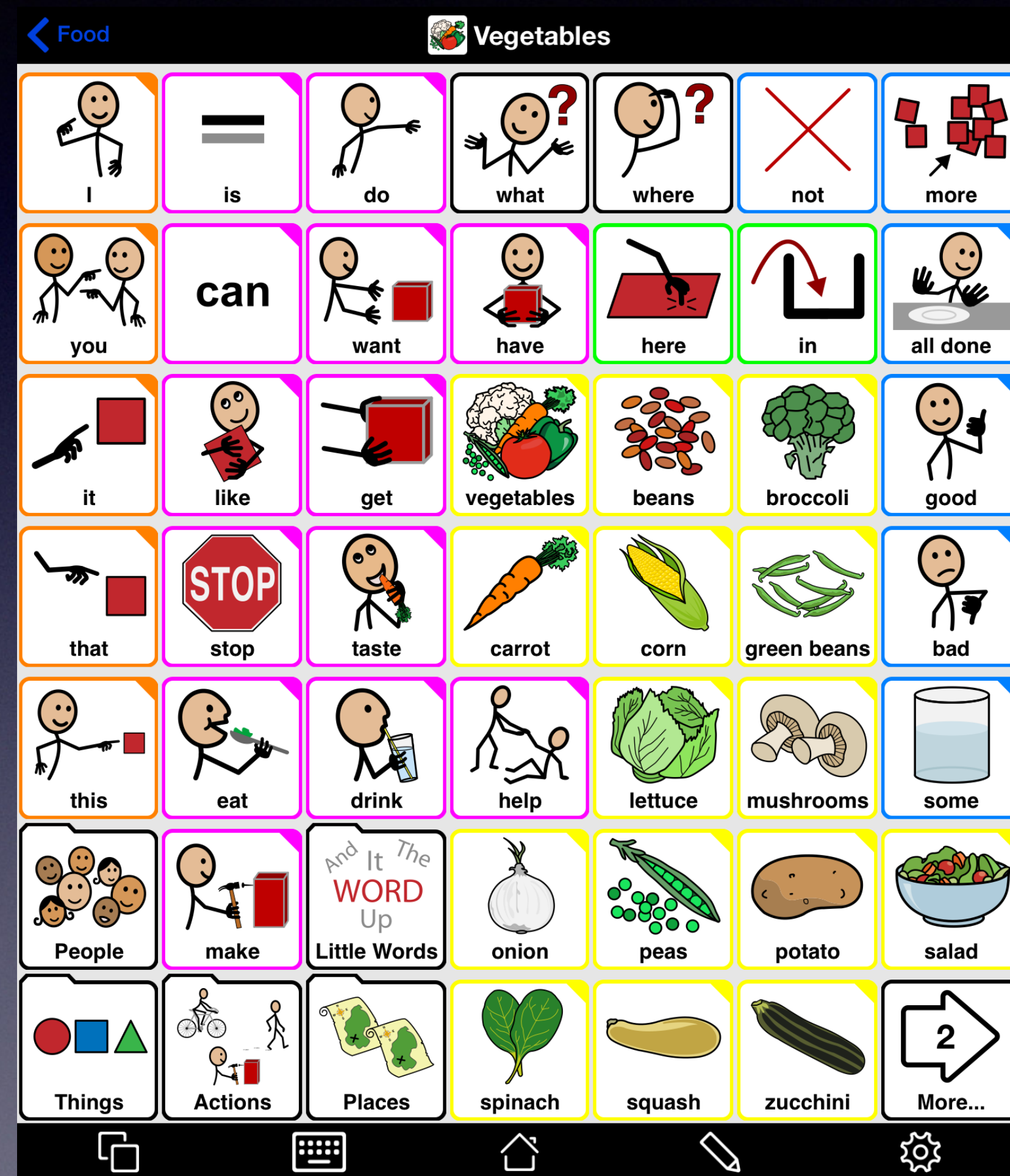
 When are we eating?	 cookie	 chips	 goldfish crackers
 fruit			

Navigation icons: copy, keyboard, home, edit, settings

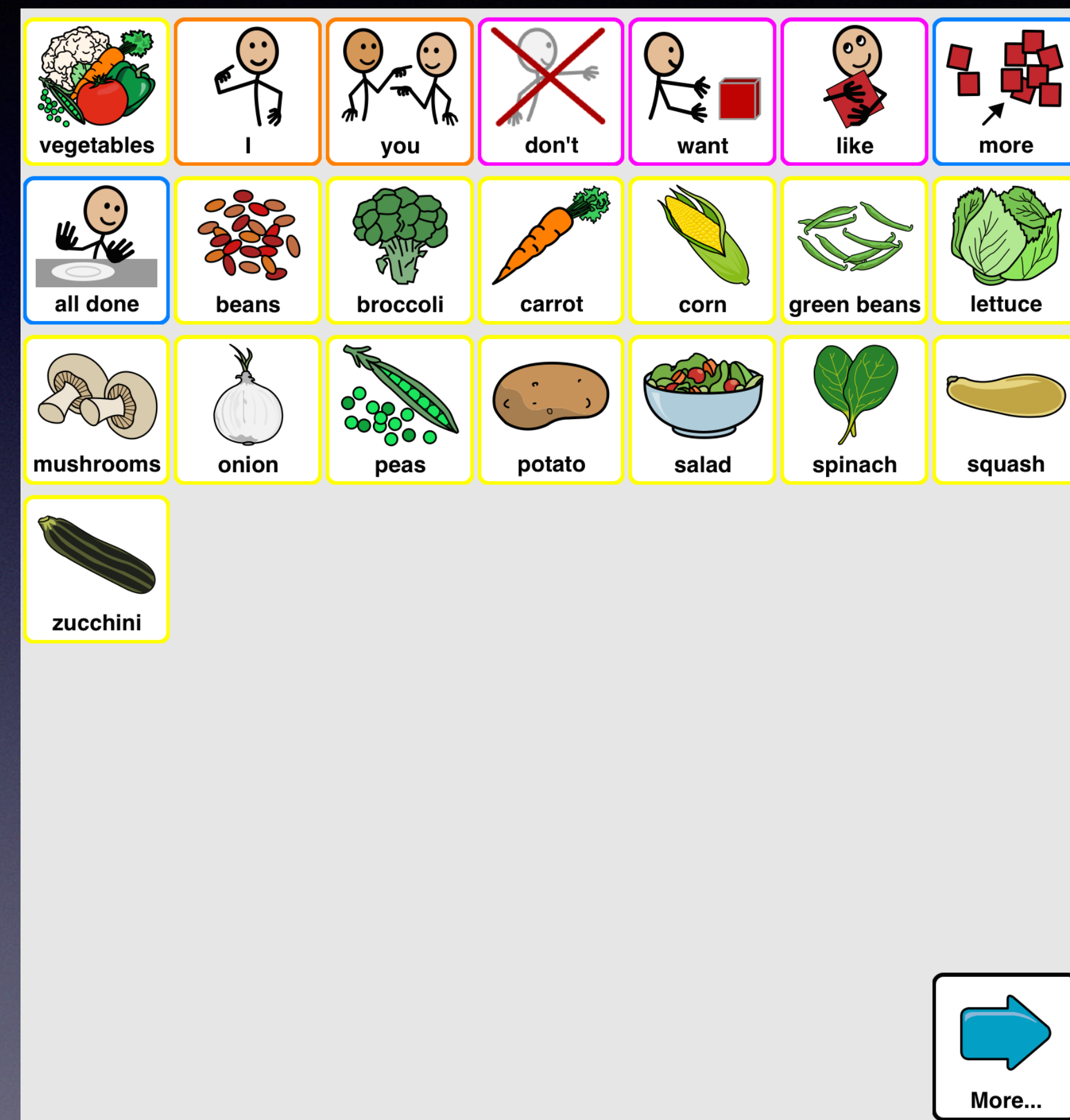
Food Template



Food Folder


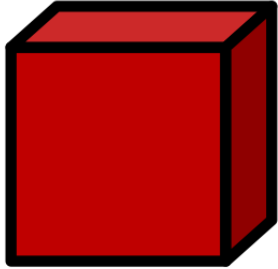
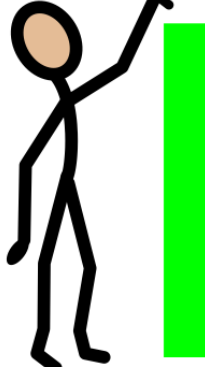

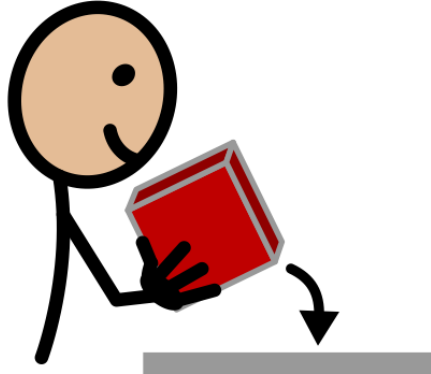
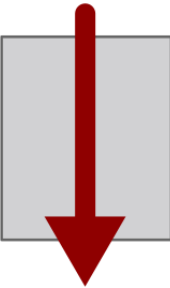
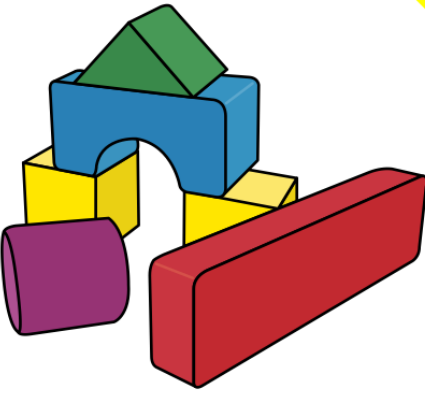
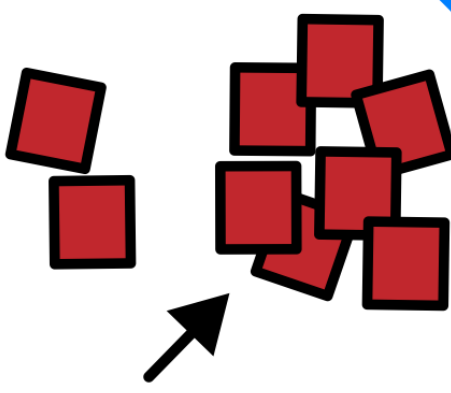
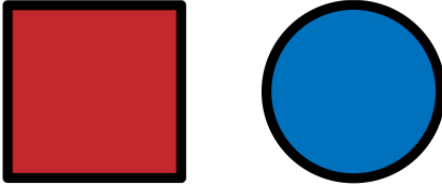


Food: Proloquo2Go 3 vs 4



Activities

[Home - 2](#) Blocks

 Let's play blocks	 I want a block	 it's so tall
 I like blocks	 put it on	 knock it down
 blocks	 want more	 different

Navigation icons: Home, Keyboard, Home, Pencil, Settings








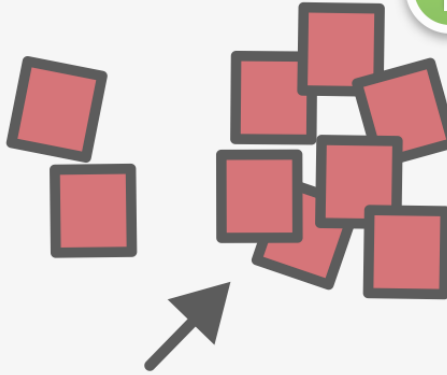


[Home - 2](#) Baby doll

 Let's play dolls	 feed	 pretty baby
 I like my baby	 hold	 change diaper
 doll	 want more	 different

Navigation icons: Home, Keyboard, Home, Pencil, Settings

Activity Template

Home Template 1 2 Done

 Let's play [topic]	 Action Phrase 1	 Adjective Phrase 1
 I like [topic]	 Action Phrase 2	 Action Phrase 3
 Topic	 want more	  different

Home Template 1 2 Done

 Let's play [topic]	 Action Phrase 1	 Adjective Phrase 1
 I like [topic]	 Action Phrase 2	 Action Phrase 3
 Topic	 want more	  different

Template button vs Placeholder button

- Template button:
 - Same for all folders that use the template
 - Marked with a T
- Placeholder button:
 - Can be different for each folder that uses the template
 - Marked with a star



www.assistiveware.com/support



Train yourself

Videos

[View detailed e-Learning videos](#)

[Watch short Support videos](#)

Tutorials & Manuals

-- Select product --

Go

Frequently Asked Questions

-- Select category --

Go

Get help

Trainers

[Find a trainer](#)

Training events

[Find a training event](#)

Resource Centers

[Find a resource center](#)

Support

[Contact us](#)

Join our community

AssistiveWare Forum

[Participate on the Forum](#)

Facebook

[Private AssistiveWare groups](#)

[AssistiveWare](#)

[Proloquo2Go](#)

[Pictello](#)

Twitter

[@assistiveware](#)

[@proloquo2go](#)